CA 3.0

ARTIKULATOR

Gebrauchsinformation

Instructions for use

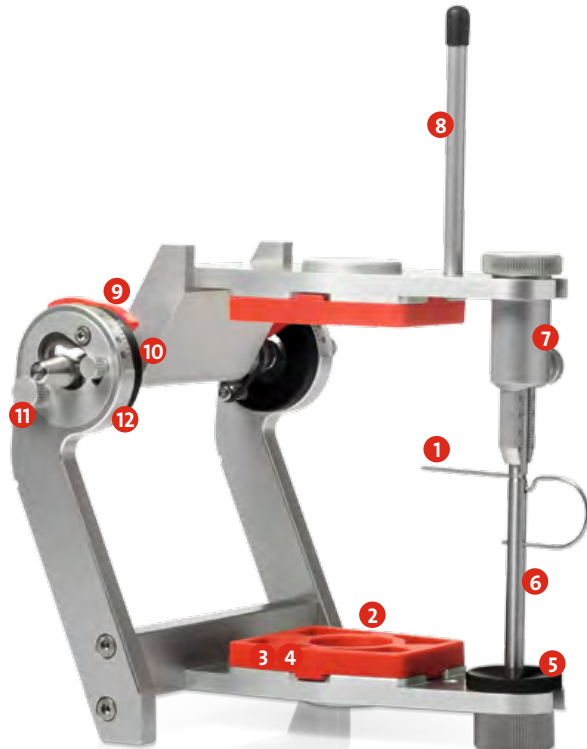
Mode d'emploi

Istruzioni d'uso

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Instructions for Use

Specifications:

- Bonwill's triangle 110 mm
- Balkwill angle 25°
- Height 150 mm, inside height 100 mm with mounting plates, 115 mm without, depth 160 mm, width 145 mm
- Retrusive path 1.5 mm
- ISS: 0-2.5 mm fixed
- Condyle path inclination adjustable 0°-60°
- Replaceable incisal guides 0°, 15° and 30°
- Weight: 690 g
- Material: anodised aluminium
- Colour: silver

1 Introduction

1.1 Preface

Thank you for purchasing a branded product from Candulor. This appliance is a technically sophisticated product. Top quality and high precision are the hallmarks of this equipment. However, risks can arise in the case of improper use. Please ensure that you have read and fully understood the instructions given below before using the product. We hope you will enjoy working with the **CA 3.0**.

1.2 Warranty

We provide a 2-year warranty from the date of purchase against material failure of original metal parts. The validity of the warranty depends on proper handling in accordance with the Instructions for Use.

2 Safety

This appliance has been developed solely for use in dentistry and must be handled strictly in accordance with the Instructions for Use. The manufacturer cannot be held liable for any damages arising as a result of incorrect handling or of using the appliance for other purposes. The user shall be solely responsible for testing the appliance prior to use in order to verify its suitability for the intended purpose, particularly if that purpose is not explicitly stated in the instructions.

3 Product Description

3.1 Function description, layout of the appliance

The **CA 3.0** is a semi-adjustable articulator designed in accordance with Camper's plane. The mechanical joint takes the form of a guide edge which supports a bi-conical axle. The horizontal condyle path inclination can be adjusted from 0°-60° by rotating a wheel. The appliance incorporates a retrusive movement of 1.5 mm as well as the possibility of an immediate side shift. A novel centric quick-lock facility can be activated for exact, reproducible initial set position of the articulator. The upper and lower sections of the articulator remain connected by the centric rubber bands even when the centric lock is open. Thanks to its well-conceived range of accessories the **CA 3.0** can be readily adapted to suit the personal preferences and requirements of the user.

3.2 Indication

Suitable for the three-dimensional fixing of models of dental reconstructions. The basic configuration is suitable for simulating the average-value movements of the human temporomandibular joint.

3.3 Contraindications

Based on current knowledge, there are no contraindications providing that the appliance is handled properly in accordance with the Instructions for Use.

4 Installation and First Use

4.1 Using the appliance for the first time

Remove the parts from the box and check for completeness. If any parts are missing or damaged, please contact Customer Service. We recommend that you keep the packaging for transportation purposes. Before using the articulator for the first time, spray with a standard silicone spray.

Do not grease!

4.2 Setting up the appliance (average value) (03)

- Loosen knurled thumb screw A and set condyle path inclination to an average value of 30°.
- Close the centric lock (04)
- Push the vertical guide rod on the vertical guide rod holder into the zero position (millimetre scale) and fasten

- Centre the incisal guide on the vertical guide rod
- Push the long part of the incisal pointer through the top hole and the short part through the bottom hole on the vertical guide rod as far as they will go.

4.3 Support rod

The supporting rod is a feature of the ergonomic design of the articulator. Screw the supporting rod into the thread in the upper section of the **CA 3.0**. When the supporting rod is fitted the upper section of the articulator will be supported (opening stop).

4.4 Mounting in the articulator (01 + 02)

The following possibilities are available for mounting in the articulator:
Direct fixing with plaster thanks to the split-cast system (01)
Plastic mounting plates, Metal mounting plates (02)

5 Handling, Operation

5.1 Centric position

The **CA 3.0** features a precision centric lock which can be activated in 3 possible positions:

Centric lock (04) closed

In this position the **CA 3.0** can only be opened and closed in centric position for occlusion checks to be performed.

Centric lock (04) open Retrusion lock closed

Protrusive, Bennett, balance movements as well as immediate side shift (ISS) and all intermediate movements can be physiologically reproduced in this position.

Centric lock and retrusion lock open (05)

With the lock in this position, the retrusive movement can also be performed.

5.2 Protrusive movement (06)

If the centric lock is open, the protrusive movement can be performed as shown in the figure. The horizontal condyle path inclination can be adjusted from 0°-60° by rotating the wheel.

5.3 Lateral and Bennett movement (07)

If the centric lock is open, the lateral movements can be performed as shown in the figure. This is done by applying backward pressure with the thumb on one side on the appropriate joint.

5.4 Immediate Side Shift (ISS) (08)

If the centric lock is open, the immediate side shift can be performed as shown in the figure. This is done by pressing in the transverse direction on one side with the index finger.

5.5 Retrusive movement (09)

A retrusive movement can only be performed if the centric lock and the retrusion lock are open. This opens the way for retrusive movement.

6 Practical Application of Model Orientation

6.1 Dentulous or edentulous case with rubber band (average value)

- Set up the appliance as described under 4.2.
- The notches in the side columns and the circular groove on the vertical guide rod can be used to orient the occlusal plane.
- Place the plaster on the mounting plate and base block, and slowly close the articulator. Next, fit a thin rubber band. Then attach it around the columns, in the notches and beneath the longer part of the incisal pointer.
- Align the occlusal plane with the rubber band.
- Align the incisal pointer between the mandibular central incisors or on the wax rim with the respective mark. Ideally, fix the lower model with modelling material and align. Finally, cast the upper model in plaster.

7 Maintenance

7.1 Cleaning

The **CA 3.0** is a precision instrument and should only be steam-cleaned. If no steam-jet equipment is available, it should be rinsed off with clean, hot water. All contact with strong acids and solvents and cleaning with ultrasonic equipment are to be avoided. Do not scrape the surface of the appliance with sharp objects.

7.2 Maintenance checks (10)

How often these checks need to be carried out will depend on the intensity of use and the user's working methods. For this reason, the stated frequency should only be taken as a guide.

- Check: for cracks and signs of damage
Part: O ring seal
Frequency: Once a month
How to replace:
- Open centric lock, and open and tighten retrusion lock
 - With the articulator open and lying on its side, slip the O ring seal over the joint axle from the outside and push over the guide edges using a blunt instrument.
 - Fit O ring seal

7.3 Repairs

The warranty will become null and void if any attempts at repair are performed within the warranty period by anyone other than a qualified Candulor service centre. Only original Candulor spare parts and accessories may be used.

8 Delivery Scope (Art. No. 72202500)

- 1 x **CA 3.0**
1 x Incisal guide 15°
1 x Incisal pointer
2 x Retention discs
2 x Plastic mounting plates
1 x Label
1 x Supporting rod for opening stop
2 x Plastering covers
1 x Instructions for use

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Candolor AG